

iQ-WEB 6.6.2 Quick Guide – Study Browser

Displays the filter for customizing the studies shown - See below

Selects all studies on this page

Double-click the "VIEWS" menu to quickly return to the EasyWEB page

Offers additional methods for searching - by Patient, Study, Series, Image, or Worklist data

Help Manual
Contains User and Admin Guides for help on features

Contains Profile settings for customizing info displayed

Click the eyeball to launch iQ-X, iQ-4VIEW or iQ-VIEW Viewer.

Begins the Search process

Click the "+" to drill into the series or image levels

Click the box to select the study

Offers additional views for studies - "EasyWEB" is the most common

	PATIENT ID	PATIENT NAME	SEX	DATE OF BIRTH	AGE	INSTITUTION NAME
<input type="checkbox"/>	MR6592	PATIENT6592	M	1901-01-01 [113]	53	GENERIC INSTITUTION
<input type="checkbox"/>	MR6486	PATIENT6486	F	1901-01-01 [113]	64	GENERIC INSTITUTION
<input type="checkbox"/>	PATID3	TEST, PDF	O			

Search
Enter a combination of criteria in the fields below the table headers to search for patients or studies. The more criteria entered, the more narrow the search will be. Wildcard characters "*" and "?" can be used for less specific matching.

Filter The "Show Filter" button above displays the Filter area where each user can customize their view. All criteria must be met for a study to display.

- Adds a quick link icon to Study Notes for the study
- Opens iQ-X or iQ-4VIEW in the current window/tab or a new one
- Determines if the Filters area closes when "Apply Filter" is clicked
- Refreshes the EasyWEB page based on the value
- Clears any Filter criteria

of studies to display at a time

Display read or unread studies

Set dates or date ranges to display

Select specific modalities

Choose specific physicians

"Date Received" is when the study was received into the PACS - NOT the Study Date

Once selections have been made, click here to apply them

iQ-WEB 6.6.2 Quick Guide – iQ-X Diagnostic Viewer

Drag series thumbnails into the panes below for viewing

Switch can be used to show/hide the thumbnail pane

Tabs show the descriptions of the various studies loaded

Retrieves uncompressed images

Loads compressed images, if possible, for faster retrieval

Double-click an image to view in a single pane. Double-click again to return to tiling.

Gives overview of active tasks

Upper bar shows progress of the image download - Lower bar, the series download

Blue border denotes the active image

Switches to next series

Image position indicator

Select to tile images from different series or studies instead of the single series

Study selection box

Series selection box

Views: SEARCH TOOLS SETTINGS COMMUNICATIONS HELP (X) root @ IQWEB

Thorax | SPINE

Hodgkin, Sabine
F 5/23/1987
ID: 00003
Acc: C0000

IMAGE Information Systems Ltd.
Dep: 7/8/2005
Time: 12:49:12

666

4cm

W4096 / C2048

CHEST

Bottom Toolbar

Buttons with a black triangle have other tools underneath. Right-click the button to access the options.

Color Remapping Scheme

Magnifier

Flip/Rotate

Tile screen

Structured Report Editor

Cine mode

Center/Windowing - Left click in the pane and drag up/down for brightness or left/right for contrast

Measurement tools

Zoom/Pan - Point to border of image for tool to zoom, middle of image for panning

Toggles the Text Overlay on and off

Resets the image

Settings - Includes "Clear Cache"