

iQ-4VIEW 1.0.0 Quick Guide – Study Browser

iQ-WEB | VIEWS | SEARCH | TOOLS | SETTINGS | COMMUNICATIONS | HELP | root @ IQWEBX

Recent | Unread | **EasyWEB** | Patients | Worklist | Jobs | Journal

Show Filter | Check All | Mark As Read | Mark As Unread | Show | Forward | Delete | Export | Download DICOM | Download C

	PATIENT ID	PATIENT NAME	SEX	DATE OF BIRTH	AGE	INSTITUTION NAME	ST
<input type="checkbox"/>	Demo008	Culare, Navi	M	2006-06-22 [9]	21	IMAGE Information Systems Ltd.	1
<input checked="" type="checkbox"/>	Demo111	Schoe, Robert	M	1940-01-01 [76]	61	IMAGE Information Systems Ltd.	1
<input type="checkbox"/>	Demo197	Nephrohepatos Cystos	M	1940-01-01 [76]	58	IMAGE Information Systems Ltd.	396

Search
Enter a combination of criteria in the fields below the table headers to search for patients or studies. The more criteria entered, the more narrow the search will be. Wildcard characters "*" and "?" can be used for less specific matching.

Help Manual
Contains User and Admin Guides for help on features

Contains Profile settings for customizing info displayed

Offers additional methods for searching - by Patient, Study, Series, Image, or Worklist data

Double-click the "VIEWS" menu to quickly return to the EasyWEB page

Selects all studies on this page

Displays the filter for customizing the studies shown - See below

Click the eyeball to launch the iQ-4VIEW viewer

Begins the Search process

Click the "+" to drill into the series or image levels

Click the box to select the study

Toggles the privacy setting of the study from Private to Public

Offers additional views for studies - "EasyWEB" is the most common

Filter The "Show Filter" button above displays the Filter area where each user can customize their view. All criteria must be met for a study to display.

Configuration:

- Entries per page: 20
- Display link to Study Notes: No Yes
- Open viewer in: Current window New window
- Always show filters: No Yes
- Autorefresh Interval: 0 seconds (min. 30)
- Clear Filter

Filter by status:

- Read Study(ies)
- Unread Study(ies)
- All

Filter by study date:

- Today
- Yesterday
- Day before yesterday
- Last 7 days
- Last 14 days
- Last 30 days
- Last 90 days
- All
- From: [] To: []

Filter by modality:

- CR CT DX MG
- MR NM OT PT
- SR RF US XA
- DOC

Filter by other:

- Reading Physician: []
- Requesting Physician: []
- Referring Physician: []
- Date Received: From: [] 00:00:00 To: [] 23:59:59
- Source AE: []
- Apply Filter

Choose specific physicians

"Date Received" is when the study was received into the PACS - NOT the Study Date

Once selections have been made, click here to apply them

Adds a quick link icon to Study Notes for the study

Opens the iQ-4VIEW viewer in the current window/tab or a new one

Determines if the Filters area closes when "Apply Filter" is clicked

Refreshes the EasyWEB page based on the value

Clears any Filter criteria

of studies to display at a time

Display read or unread studies

Set dates or date ranges to display

Select specific modalities

iQ-4VIEW 1.0.0 Quick Guide – Diagnostic Viewer

Closes viewer and opens the last used navigation

Maximizes the browser window. Press [ESC] to exit the full screen mode.

Drag series thumbnails into the panes below for viewing

Opens a menu to adjust settings of the viewer

Gives information about the software version and contact details

Opens user documentation in new tab as PDF

Offers drop-down box to change the user interface language

Switch can be used to show/hide the series preview bar

Switch can be used to pin/unpin the series preview bar

Opens second series preview bar for second study loaded

Double-click an image to view in a single pane. Double-click again to return to tiling.

Tabs show the descriptions of the various studies loaded

Orange border denotes the active image

Bottom Toolbar Right-click the button (or tap and hold on touch screens) to access the options.

Cine mode

Center/Windowing - Left click in the pane and drag up/down for brightness or left/right for contrast

Color Remapping Scheme

Measurement tools

Magnifier

Zoom/Pan - Point to border of image for tool to zoom, middle of image for panning

Flip/Rotate

Tile screen on series level

Tile screen on image level

Toggles the Text Overlay on and off

Structured Report Editor

Activates scoutlines

Activates synchronization

Resets the image

Switch can be used to show/hide the series preview bar

Switch can be used to pin/unpin the series preview bar