

# iQ-4VIEW 2.0 Quick Guide – Study Browser

**iQ-WEB** VIEWS SEARCH TOOLS SETTINGS COMMUNICATIONS HELP root @ IQWEBX

Recent Unread **EasyWEB** Patients Worklist Jobs Journal

Show Filter Check All Mark As Read Mark As Unread Show Forward Delete Export Download DICOM Download C

	PATIENT ID	PATIENT NAME	SEX	DATE OF BIRTH	AGE	INSTITUTION NAME	ST
<input type="checkbox"/>	Demo008	Culare, Navi	M	2006-06-22 [9]	21	IMAGE Information Systems Ltd.	1
<input checked="" type="checkbox"/>	Demo111	Schoe, Robert	M	1940-01-01 [76]	61	IMAGE Information Systems Ltd.	1
<input type="checkbox"/>	Demo197	Nephrohepatos Cystos	M	1940-01-01 [76]	58	IMAGE Information Systems Ltd.	396

**Search**  
Enter a combination of criteria in the fields below the table headers to search for patients or studies. The more criteria entered, the more narrow the search will be. Wildcard characters "\*" and "?" can be used for less specific matching.

**Help Manual**  
Contains User and Admin Guides for help on features

Contains Profile settings for customizing info displayed

Offers additional methods for searching - by Patient, Study, Series, Image, or Worklist data

Double-click the "VIEWS" menu to quickly return to the EasyWEB page

Selects all studies on this page

Displays the filter for customizing the studies shown - See below

Click the eyeball to launch the iQ-4VIEW viewer

Begins the Search process

Click the "+" to drill into the series or image levels

Click the box to select the study

Toggles the privacy setting of the study from Private to Public

Offers additional views for studies - "EasyWEB" is the most common

**Filter** The "Show Filter" button above displays the Filter area where each user can customize their view. All criteria must be met for a study to display.

Configuration: # of studies to display at a time

Entries per page: 20

Display link to Study Notes:  No  Yes

Open viewer in:  Current window  New window

Always show filters:  No  Yes

Autorefresh Interval: 0 seconds (min. 30)

Clear Filter

Filter by status: Display read or unread studies

Read Study(ies)  
 Unread Study(ies)  
 All

Filter by study date: Set dates or date ranges to display

Today  
 Yesterday  
 Day before yesterday  
 Last 7 days  
 Last 14 days  
 Last 30 days  
 Last 90 days  
 All

From: To:

Filter by modality: Select specific modalities

CR  CT  DX  MG  
 MR  NM  OT  PT  
 SR  RF  US  XA  
 DOC

Filter by other: Choose specific physicians

Reading Physician  
Requesting Physician  
Referring Physician

Date Received: "Date Received" is when the study was received into the PACS - NOT the Study Date

From: 00:00:00 To: 23:59:59

Source AE

Apply Filter

Once selections have been made, click here to apply them

Adds a quick link icon to Study Notes for the study

Opens the iQ-4VIEW viewer in the current window/tab or a new one

Determines if the Filters area closes when "Apply Filter" is clicked

Refreshes the EasyWEB page based on the value

Clears any Filter criteria

# iQ-4VIEW 2.0 Quick Guide – Diagnostic Viewer

Closes viewer and opens the last used navigation

Maximizes the browser window. Press [ESC] to exit the full screen mode.

Opens a menu to adjust important user settings, e.g. select the client mode for small studies on fast clients or the diagnostic server mode for large studies on slow client computers.

Gives information about the software version and contact details

Opens user documentation in new tab as PDF

Offers drop-down box to change the user interface language

Switch can be used to show/hide the series preview bar

Switch can be used to pin/unpin the series preview bar

Opens second series preview bar for second study loaded for image comparison

Double-click an image to view in a single pane. Double-click again to return to tiling.

Drag series thumbnails into the panes below or double click them for viewing

Tabs show the descriptions of the various studies loaded. Select a tab to open a study.

Orange border marks the active image

## Bottom Toolbar Left-click to select a function. Right-click to access options (On touch screen devices tap and hold instead of a right click).

Stack/Cine mode

Center/Windowing - Left click in the pane and drag up/down for brightness or left/right for contrast

Measurement tools

Color Remapping Scheme

Magnifier Flip/Rotate

Zoom/Pan - Point to border of image for tool to zoom, middle of image for panning

Determines if changes apply to the current image or series

Tile screen on image level

Tile screen on series level

Toggles the Text Overlay on and off

Structured Report Editor

Activates synchronization

Resets the image

Activates scoutlines

Switch can be used to show/hide the series preview bar

Switch can be used to pin/unpin the series preview bar