

# iQ-4VIEW 2.1 Quick Guide – Study Browser

**iQ-WEB** VIEWS SEARCH TOOLS SETTINGS COMMUNICATIONS HELP root @ IQWEBX

Recent Unread **EasyWEB** Patients Worklist Jobs Journal

Show Filter Check All Mark As Read Mark As Unread Show Forward Delete Export Download DICOM Download C

	PATIENT ID	PATIENT NAME	SEX	DATE OF BIRTH	AGE	INSTITUTION NAME	ST
<input type="checkbox"/>	Demo008	Culare, Navi	M	2006-06-22 [9]	21	IMAGE Information Systems Ltd.	1
<input checked="" type="checkbox"/>	Demo111	Schoe, Robert	M	1940-01-01 [76]	61	IMAGE Information Systems Ltd.	1
<input type="checkbox"/>	Demo197	Nephrohepatos Cystos	M	1940-01-01 [76]	58	IMAGE Information Systems Ltd.	396

**Search**  
Enter a combination of criteria in the fields below the table headers to search for patients or studies. The more criteria entered, the more narrow the search will be. Wildcard characters "\*" and "?" can be used for less specific matching.

**Help Manual**  
Contains User and Admin Guides for help on features

Contains Profile settings for customizing info displayed

Offers additional methods for searching - by Patient, Study, Series, Image, or Worklist data

Double-click the "VIEWS" menu to quickly return to the EasyWEB page

Selects all studies on this page

Displays the filter for customizing the studies shown - See below

Click the eyeball to launch the iQ-4VIEW viewer

Begins the Search process

Click the "+" to drill into the series or image levels

Click the box to select the study

Toggles the privacy setting of the study from Private to Public

Offers additional views for studies - "EasyWEB" is the most common

**Filter** The "Show Filter" button above displays the Filter area where each user can customize their view. All criteria must be met for a study to display.

Configuration: # of studies to display at a time

Entries per page: 20

Display link to Study Notes:  No  Yes

Open viewer in:  Current window  New window

Always show filters:  No  Yes

Autorefresh Interval: 0 seconds (min. 30)

Clear Filter

Filter by status: Display read or unread studies

Read Study(ies)  
 Unread Study(ies)  
 All

Filter by study date: Set dates or date ranges to display

Today  
 Yesterday  
 Day before yesterday  
 Last 7 days  
 Last 14 days  
 Last 30 days  
 Last 90 days  
 All

From: To:

Filter by modality: Select specific modalities

CR  CT  DX  MG  
 MR  NM  OT  PT  
 SR  RF  US  XA  
 DOC

Filter by other: Choose specific physicians

Reading Physician  
Requesting Physician  
Referring Physician

Date Received: "Date Received" is when the study was received into the PACS - NOT the Study Date

From: 00:00:00 To: 23:59:59

Source AE

Apply Filter

Once selections have been made, click here to apply them

Adds a quick link icon to Study Notes for the study

Opens the iQ-4VIEW viewer in the current window/tab or a new one

Determines if the Filters area closes when "Apply Filter" is clicked

Refreshes the EasyWEB page based on the value

Clears any Filter criteria

# iQ-4VIEW 2.1 Quick Guide – Diagnostic Viewer

**Navigation and Window Management:**

- Drag series thumbnails into the panes below or double click them for viewing
- Maximizes the browser window. Press [ESC] to exit the full screen mode.
- Closes viewer and opens the last used navigation

**Study Selection and Comparison:**

- Orange border marks the active image
- Switch can be used to show/hide the series preview bar
- Switch can be used to pin/unpin the series preview bar
- Offers drop-down box to change the user interface language
- Opens user documentation in new tab as PDF
- Opens second series preview bar for second study loaded for image comparison

**Settings and Information:**

- Opens a menu to adjust important user settings, e.g. select the client mode for small studies on fast clients or the diagnostic server mode for large studies on slow client computers.
- Gives information about the software version and contact details

**Image Interaction:**

- Double-click an image to view in a single pane. Double-click again to return to tiling.

## Bottom Toolbar Left-click to select a function. Right-click to access options (On touch screen devices tap and hold instead of a right click).

**Bottom Toolbar Functions:**

- Stack/Cine mode**: Icon of a film strip.
- Center/Windowing**: Left click in the pane and drag up/down for brightness or left/right for contrast.
- Color Remapping Scheme**: Icon of a color wheel.
- Measurement tools**: Icon of a ruler.
- Magnifier**: Icon of a magnifying glass.
- Zoom/Pan**: Point to border of image for tool to zoom, middle of image for panning.
- Flip/Rotate**: Icon of a hand with a circular arrow.
- Determines if changes apply to the current image or series**: Icon of a person with a checkmark.
- Tile screen on image level**: Icon of a 2x2 grid.
- Tile screen on series level**: Icon of a 2x2 grid with a plus sign.
- Toggles the Text Overlay on and off**: Icon of a 'T' in a circle.
- Structured Report Editor**: Icon of a document with a pencil.
- Creates DICOM Secondary Captures**: Icon of a camera.
- Activates scoutlines**: Icon of a person silhouette.
- Activates synchronization**: Icon of two linked circles.
- Resets the image**: Icon of a circular arrow.
- Switch can be used to show/hide the series preview bar**: Icon of a series preview bar.
- Switch can be used to pin/unpin the series preview bar**: Icon of a pin.